

Model 109 WALKING AID TROLLEY

Introduction

The Days Healthcare 109 Walking Aid Trolley is manufactured from powder coated steel tubing with four solid tyre wheels. It incorporates 2 trays and 2 adjustable handles with locking brakes.

This walking aid is intended for indoor use only. It is designed to carry light loads in a domestic environment, whilst providing support for the user.

To fit the trays to the frame, locate the lugs under the tray under the cross strap and gently push the tray down onto the tubular frame at the front. The tray will clip into position.



Handle Adjustment

Remove the hand knob and bolt.

Pull the handle to new setting (there are six fixed settings at 25 mm intervals), re-fit and tighten the knob and bolt ensuring that the head of the bolt is inside the hexagonal hole.

Operation

The 109 Walking Aid trolley is fitted with locking brakes. To apply the brakes pull up on the brake handles. To lock the brakes push down on both brake handles until they click into the locked position. To release the brakes simply pull up on the brake handles until they click to unlock.

If the user finds the brakes hard to apply, the pressure can be adjusted by reducing the tension in the spring above the brake shoe. Hold the hexagonal nut and turn the knurled knob to vary the brake pressure i.e. the distance between the shoe and the tyre, when the brakes are off. Both the knob and nut are located above the brake shoe. It is a case of trial and error to suit the individual users' grip strength.

- ❗ **WARNING!** - Brakes that are adjusted to give light pressure will be less effective at preventing rear wheel movement under load.

When using the walking aid trolley be aware that stability depends on all four wheels being in contact with the floors flat surface.

Cleaning & Maintenance

The Days Healthcare Walking Aid Trolley is designed for minimum maintenance. It is manufactured from powder coated steel tubing. To clean, wipe over with a damp soft cloth. Use general purpose neutral detergent and warm water. Do not use bleach based or harsh abrasives.

To disinfect clean as above and wipe over with solution of one thousand parts per million available chlorine disinfectant (haztabs/precept), rinse and dry.

To remove the trays for washing or steam cleaning, simply unclip at the front and lift to remove.

Daily, before use, check the effective operation of the brakes. Check that the wheels turn and swivel freely and there is no debris building around the axle. Periodically check that all screws, nuts and bolts are securely fastened. Check moving parts for wear.

Specifications

- ✦ Maximum user weight: 125 Kg
- ✦ Handle height adjustment: 760 – 910 mm
- ✦ Overall width: 600mm
- ✦ Overall length: 550 mm
- ✦ Overall weight: 8 Kg

Warranty Information

The Days Healthcare Model No.109 is warranted for one year from the date of purchase against faulty workmanship or materials. Please contact your supplier/dealer should a fault occur.

The warranty does not extend to the consequential costs resulting from fault clearance, in particular freight and travel costs, loss of earnings, expenses, etc.

The manufacturer will not accept responsibility for any damage or injury caused by misuse or non-observance of the instructions set out above.